



COMPEX, spol. s r.o.

A/ Palackého 105, c. p. 924, 612 00 Brno T/ +420 606 071 596 E/ export@jett.eu / W W W . J E T T . E U

JETT Plasma Lift

Experiences from JETT Plasma Lift Nederland





COMPEX, spol. s r.o.

A/ Palackeho 105, c. p. 924, 612 00 Brno T/ +420 606 071 596 E/ export@jett.eu / W W W . J E T T . E U

Belly Treatment

- Apply ultrasound gel
- 1) Flat Applicator → 30 min
 - 2) 5 mm Applicator → 10 min
 - 3) 3 mm Applicator → 10 min
 - 4) Cone Applicator → apply JETT LiftVital Serum → 5 min scanning with plasma spark






COMPEX, spol. s r. o.

A/ Palackého 105, c. p. 924, 612 00 Brno T/ +420 606 071 596 E/ export@jett.eu / W W W . J E T T . E U

Thighs Treatment

- Apply ultrasound gel
- 1) Flat Applicator → 30 min
 - 2) 5 mm Applicator → 10 min
 - 3) 3 mm Applicator → 10 min
 - 4) Cone Applicator → apply JETT LiftVital Serum → 5 min scanning with plasma spark

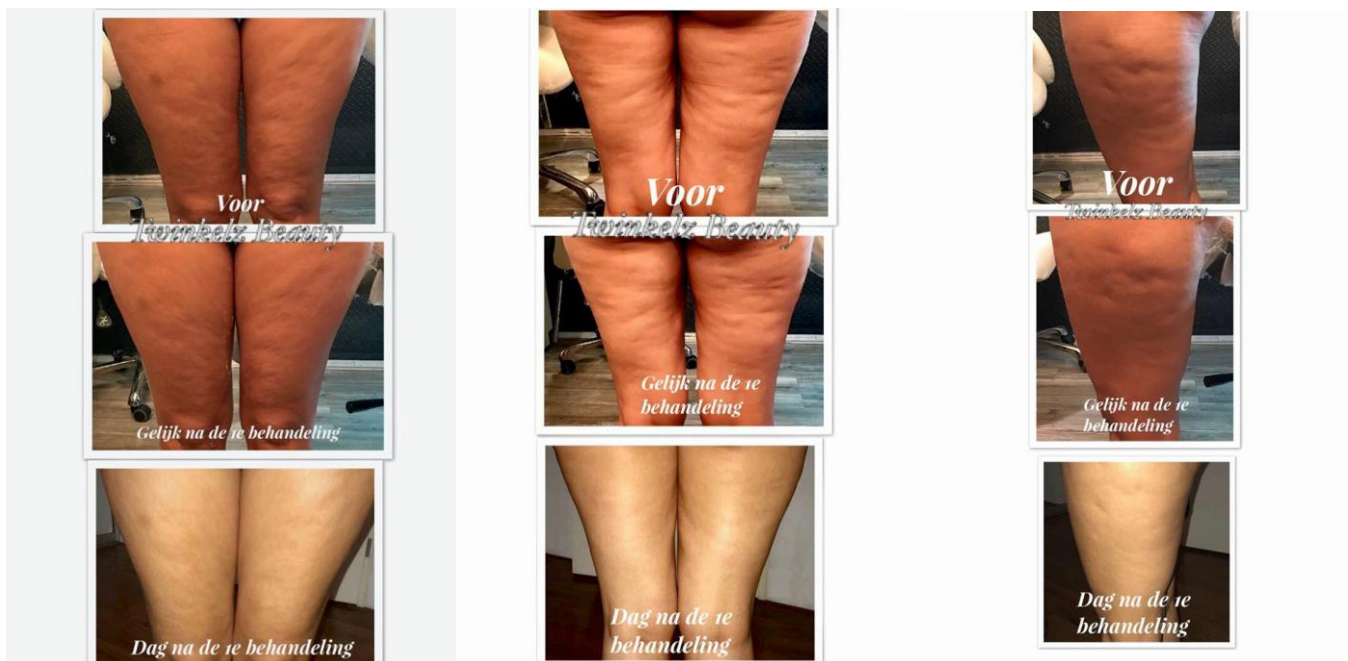



COMPEX, spol. s r.o.

A/ Palackého 105, c. p. 924, 612 00 Brno T/ +420 606 071 596 E/ export@jett.eu / W W W . J E T T . E U

Face Treatment

- 1) Removal of make up and cleaning of the skin
- 2) Peeling
- 3) Massage of the face
- 4) Flat Applicator → apply ultrasound gel → 15-20 min
- 5) 5 mm Applicator → apply ultrasound gel → 10 min (back and forth movement, zig-zag movement over the wrinkle)
- 6) 3 mm Applicator → apply ultrasound gel → 10 min (circle movement over the wrinkle)
- 7) Cone Applicator → apply think layer of Jett Liftvital Serum → highest intensity → 5 min → scanning of the wrinkle





COMPEX, spol. s r.o.

A/ Palackého 105, c. p. 924, 612 00 Brno T/ +420 606 071 596 E/ export@jett.eu / W W W . J E T T . E U

Upper Eyelid Lifting

- 1) Flat Applicator → apply ultrasound gel → 10 min
- 2) 3 mm Applicator → apply ultrasound gel → 8 min → circle movement
- 3) Cone Applicator → apply JETT Liftvital Serum → 5 min





COMPEX, spol. s r. o.

A/ Palackého 105, c. p. 924, 612 00 Brno T/ +420 606 071 596 E/ export@jett.eu / W W W . J E T T . E U

Upper Lip Lifting

- Apply ultrasound gel on lip
- 1) Flat Applicator → 10 min
- 2) Cone A applicator → 5 min of scanning with plasma spark
- 3) After apply Jett Liftvital Serum