

JETT PLASMA LIFT MEDICAL

PLASMA PEN FOR OPHTHALMOLOGISTS



- Plasma Pen for a medical device JETT PLASMA LIFT MEDICAL is a lightweight ergonomic extension of the main device JETT PLASMA LIFT MEDICAL.
- The pen is easy to use by attaching it instead of an applicator to a main core of the device. Thus, you can treat the skin with the Plasma Pen only while the main core stays on the stand.
- Available manual operation right on the pen ensures easy use and mobility.
- The pen is equipped with 7 applicators – 5 silver applicators for a cosmetic contact treatment and 2 golden applicators for mini-invasive medical procedures.
- Plasma Pen is compatible with all types of the device JETT PLASMA LIFT MEDICAL.



CE 0123



JETT PLASMA LIFT MEDICAL

PLASMA PEN FOR OPHTHALMOLOGISTS



MAIN INDICATIONS

- Blepharitis
- Ectropion
- Trichiasis
- Xanthelasma
- Fibroma removal in periorbital area
- Aesthetic scar treatment
(e.g. after surgical blepharoplasty)

MECHANISM OF ACTION

Blepharitis

- Release of Meibomian gland margins
- Softening of gland content, loosening and liquefaction of sebum and removal from the glands thanks to the plasma generated heat
- Anti-inflammatory effect – thrombosis of the vascular system round the Meibomian glands may play a role in reducing the local release of inflammatory mediators
- Elimination of the bacterial flora in the glands and their surroundings
- Increase of elasticity of the ligament fibres round the glands and their outlets (due to DC current)
- Plasma helps to increase production of the non-inflamed Meibomian glands

Ectropion

- Carbonization of tissue, its contraction and return to its original position

Non-invasive blepharoplasty



Before

After

Ectropion



Before

After

After

Smart non-invasive solution not only for patients with blepharitis

Plasma, the fourth state of matter created by ionized gas, has a wide range of applications in ophthalmology.

Thanks to its patented technology, the JETT PLASMA LIFT MEDICAL device generates plasma by direct current. Unlike the devices utilizing alternate current, this technology allows to generate a stable stream facilitating precise targeting of the discharge.



OPHTHALMOLOGICAL SET

- JETT PLASMA LIFT MEDICAL device
- lightweight Plasma Pen
- set of ophthalmological applicators
- metal stand